COMFORTISSE® POSTURE PRO

EN

Comfortisse® Posture Pro helps you achieve an upright posture effortlessly.

Warnings:

- Wash Comfortisse® Posture Pro before using it for the first time to reduce any odours. Odours will diminish over time if the product is kept well-ventilated.
- Comfortisse® Posture Pro should fit snugly without being uncomfortable. If you find Comfortisse® Posture Pro to be uncomfortable, loosen the straps.
- Make sure you can breathe deeply while wearing Comfortisse® Posture Pro.
- Do not use Comfortisse® Posture Pro while sleeping.
- Stop wearing the garment if your skin breaks out or itches.

Product Description

1. Belt.
2. Elastic straps.

Instructions for Use:

1. Unfasten the belt.
2. Loosen the elastic straps.
3. Put your arms in (A).
4. Adjust the belt to make it as tight and comfortable as possible (B).
5. Stick the end of the belt in the correct position.
6. Adjust the elastic straps until you achieve a comfortable, upright posture (C).
7. Pass the ends of the elastic straps through the fasteners. If you wish, you can cut off any excess strap.

Cleaning instructions:

- Wash by hand in cold water.
- Do not use bleach.
- Do not tumble dry; allow to air dry.
- Do not expose to the sun.
- Do not iron.
- Do not wring.
- Do not dry clean.

Composition:

Main fabric:
 Inner layer: 92% polyester, 8% hemp fibre
 Middle layer: 100% SBR (styrene-butadiene rubber)
 Outer layer: 100% polyester
 Elastic band: 82% polyester, 18% elastodiene
 Piping: 90% polyester, 10% elastane
 Ribbon: 100% polyester
 Fastener: 60% polyester, 40% nylon

Quality Warranty

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country.
 This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

Australia & New Zealand only

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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